

INTENDED USE

- The InTec Rapid SARS-CoV-2 Antigen Test is a colloidal gold enhanced rapid immunochromatographic assay for qualitative detection of nucleocapsid protein antigen from SARS-CoV-2.
- This test is authorized for home use with self-collected anterior nasal swab samples from individuals aged 18 years or older with symptoms of COVID-19 within the first seven (7) days of symptom onset.






SPECIFICATION

Specimen Type	Result Availability	Shelf Life	Storage Condition	
Nasal swab	15-20 minutes	Up to 24 months	2-30°C	

CLINICAL SIGNIFICANCE

When your COVID-19 test will be positive

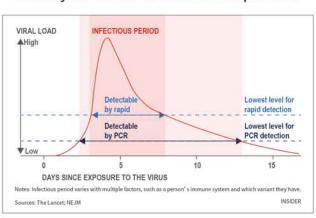

PERFORMANCE CHARACTERISTICS

		PCR		
		Positive	Negative	Total
lies - STARWINGS STRIKE	Positive	103	0	103
Rapid SARS-CoV-2 Antigen Test	Negative	7	470	477
/ wagen resc	Total	110	470 5	580

Results analysis:

Sensitivity: 103/110=93.64% Specificity: 470/470=100.00%

Total coincidence rate: 573/580=98.79%


TEST PROCEDURE

Prepare Materials

- 1. Peel off the foil from the top of the buffer tube.
- 2. Set the buffer tube on the stand hole of the package. For package of 5/7/25 tests, tube holder is provided.
- 3. Remove the swab from its package, being careful not to touch the tip of the swab.






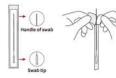


Collect Sample

- 1. Gently insert the entire absorbent tip of the swab (usually 2.0 to 2.5cm) into your nostril.
- 2. Rotate the swab 5 times against the nasal wall and remove the swab.
- 3. Using the same swab, repeat the same sample collection procedure for the other nostril.

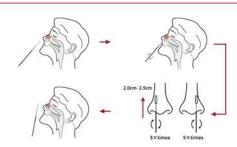

Process Sample

- 1. Insert the swab into the tube, touch the bottom of the tube with the swab tip, and stir at least 10 times.
- 2. Squeeze the tip of the swab along the inner tube wall and then remove and discard it.
- 3. Press the nozzle cap tightly onto the buffer tube to avoid any leaks.

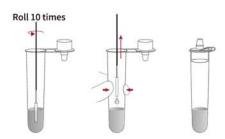

Add Sample


- 1. Unseal the foil pouch and put the cassette on a clean, dry and level surface.
- 2. Add 5 drops (~100µl) of treated specimen into "S" well of the cassette.
- 3. Wait at least 15 minutes (20 minutes at most) to interpret result.







Read Result

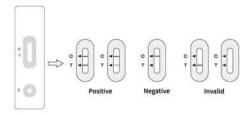

ORDERING INFORMATION

Product	Cat. No.	Туре	Pack Size	Specimen	Certification
Rapid SARS-CoV-2 Antigen Test for self-testing	ITP16030-TC1	Cassette	1T	Nasal swab	CE
	ITP16030-TC2	Cassette	2T	Nasal swab	CE
	ITP16030-TC5	Cassette	5T	Nasal swab	CE
	ITP16030-TC7	Cassette	7T	Nasal swab	CE
	ITP16030-TC25	Cassette	25T	Nasal swab	CE

